

PROTEC BATHS

CRAFTED IN BRITAIN

Chromotherapy LED lighting system for Assisted Baths Manual



Tel. 01726 438694 Email. sales@protecbaths.com

www.PROTECBATHS.com

PROTEC BATHS

CRAFTED IN BRITAIN

The Chromotherapy Lighting system has 7 individual colours, and also has the option to run through a programme of colour changing sequences, all operated by the touch buttons on the top of the bath.

The lights are securely fastened through the surface of the bath and are the housing design is fully waterproof. The housing is IP rated and operates via a low voltage supply, so the system is safe to use within and around a bath.

Specifications:

- Rainbow Spectrum of Rotating Lights and Green, Orange, Red, Violet, Blue, Yellow, Indigo Solid Colour
- LED Lamp 8000 hours usage this equates to approx. 11 years of daily use
- CE, IPX8 Front and IPX4 back – IP68
- Illuminated touch sensitive button – controller/on/off
- AC220V – 240V to AC12V Transformer
- 3 Watt

PROTEC BATHS

CRAFTED IN BRITAIN

Chromotherapy LED lighting system User Guide

Press these buttons to turn on, change colour and turn off:

(If there are more than one button, one will control the lights at the seat end of the bath, the other will control the ones at the tap end of the bath)



**** The buttons are touch sensitive, they do not require force to turn on and off – pressing these hard could cause damage ****

Instructions for Use:

- Press button once to turn on
- Press again to change to the next colour (colours start at Green)
- Press again to change to next colour
- Do this to change through all 7 colours

PROTEC BATHS

CRAFTED IN BRITAIN

- At the end of the sequence the last press will turn them off
- At any time in the sequence, press and hold for 2 seconds to change to slow auto rotating spectrum of colours

Cleaning:

The Lights & touch buttons may be cleaned using mild cleaning agents and anti-bacterial solutions.

Chemical agents and solvents must not be used, for example, Drain/oven Cleaner, Acetone, or Acids.

Do not use abrasive cleaning cloths, sponges, or scourers to clean these items of the Bath or tapware.

If you have any queries, please call us on
01726 438694